Ampollas de Gas esterilizante (EO)

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Sterilizing Gas (Ethylene Oxide) Ampoules

Product Overview

This product includes a kit that contains:

- EO sterilization gas ampoule as per presentation.
- A membrane container for diffusion.
- Containment bag for the placement of materials designated for sterilization (sizes vary based on presentation).
- Sealing mechanism for the bag

Features

Ethylene oxide is a colorless gas that is highly effective in sterilizing heat-sensitive materials, including plastics, textiles, and medical devices. Its penetrating properties eradicate bacteria, viruses, fungi, and spores by altering the DNA and proteins of microorganisms, thereby disrupting their life cycle.

User-friendly in various settings, including emergencies, due to its sterilization capability at ambient temperature and pressure.

Presentations

IF				
CODE	BOTTLE	MEASURES	ABILITY	AMOUNT
4005	5 cc	31 x 60 cm	11 lts	20
4020	20 cc	56 x 96 cm	34 lts	5
4320	EO4320 Sterilizing Gas Cartridge (Kit with Humidity)			
4020-A	20 cc	56 x 96 cm	34 lts	10*

→ Exclusive sales presentation for global shipments.

Rev03-05.06.25

Intended purpose

An ampoule engineered to release ethylene oxide during sterilization processes, ideally within airtight chambers. It is particularly suited for sterilizing thermosensitive materials at ambient temperature and pressure, in hospital or emergency environments.

How to Utilize

Place the ampoule within the container bag alongside the material designated for sterilization. Donning safety gloves, break the ampoule to ensure the complete release of the gas within the bag. Subsequently, initiate the appropriate sterilization cycle.

Precautions

Prior to handling this product, consult the Material Safety Data Sheet (MSDS) for comprehensive information regarding hazards, safe handling procedures, and suitable protective measures.

Storage

Store in a cool, dry location, shielded from and protected against direct light.

Useful Life

Shelf life of 36 months from the date of production, provided that the specified storage conditions are maintained.

Regulation

All activities conducted at EFELAB adhere to the ISO 13485 quality standard and comply with Good Manufacturing Practices (GMP) to guarantee product quality.

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