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## Medical-Grade Crepe Paper | TP02

### Features

Bowie-Dick test packs are meticulously crafted for the daily monitoring of vacuum pumps in steam sterilizers and pre-vacuum cycles.

### Product Overview

This indicator is intended to assess the efficacy of the air removal system and steam penetration in pre-vacuum steam sterilizers. The chemical indicator alters its color solely when the critical parameters of time, temperature, and saturated steam are satisfied, thereby simulating a genuine load. Its design creates a controlled resistance to the flow of steam, ensuring a precise evaluation of the team performance.

***Exclusive access for healthcare professionals and institutions.***

## Presentation

The product is offered in individual packaging, and the test sheet dimensions are 108 mm x 119 mm.

## How to Utilize

Expose the chemical indicator to a temperature of 121 °C for 15 minutes, or to a temperature of 134 °C for 15 minutes.

3.5-minute exposure (+/- 30 seconds).

An external indicator that enables verification of the packet's processing status. A successful outcome is indicated by a uniform and complete color change on the indicator, confirming effective air evacuation and sufficient steam penetration. Conversely, a poor outcome is characterized by areas of colorlessness or partial discoloration, suggesting potential residual air or a malfunction in the sterilizer's vacuum system.

The alteration in color of the sheet printed with sensitive ink is irreversible and can be preserved as a record of the sterilization cycle.

## Precautions

This product should not be utilized for any processes other than moist heat in a steam sterilizer.

## Storage

Store in a cool, dry location, shielded from and protected against direct light. Refrain from storing in areas where other chemical agents are present.

## Useful Life

Shelf life: 60 months from the date of manufacture, provided the specified storage conditions are maintained. This product is not intended for reuse.

## Regulation

Engineered in accordance with ISO 11140-1, 4

All activities conducted at EFELAB adhere to established quality standards. ISO 13485, along with adherence to Good Manufacturing Practices, to guarantee the product excellence.

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