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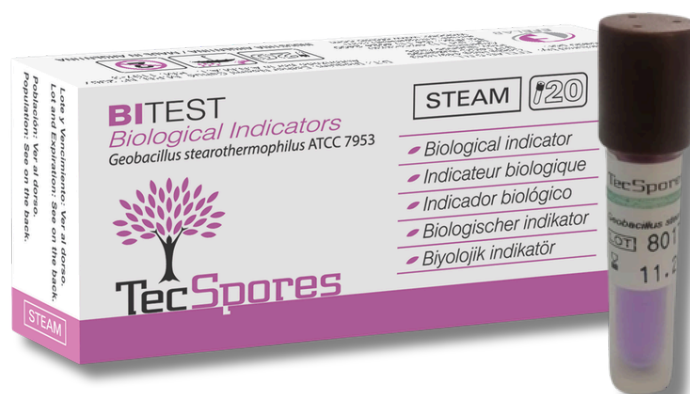
Self-Contained Biological Indicators For Humid Heat.

Product Overview

The TecSpores® Biological Indicator assesses sterilization efficacy through the use of spores *Geobacillus stearothermophilus* and a medium that transitions to a translucent yellow upon the detection of bacterial growth.

Features

The strain *Geobacillus stearothermophilus* is biocompatible for monitoring steam sterilization. Its resistance to this process has established it as a representative organism for sterilization control.

Presentations

Self-Contained Biological Indicator Box: 100 units (BIS-C5 and BIS-C6) and 20 units (BIS-C520 and BIS-C6)

Exclusive access for healthcare professionals and institutions.

Rev05-14.04.26

   EFELAB S.R.L.

 (+5411) 4623-3030 | 4624-2400 | 4747-0000

 (+54) 11-5977-6578 |  www.efelab.com |  info@efelab.com

Intended purpose

Position the biological indicator(s) (BI) within the sterilizer in accordance with established best practices, conduct the sterilization process as standard, and manage the BI after a duration of 10 minutes. Confirm the color change of the chemical indicator. Activate the BI alongside an unprocessed BI (serving as a positive control). Incubate for 24 hours at a temperature of 55-60°C, ensuring it remains upright throughout the incubation period. Review the results every 12 hours; upon obtaining a positive result, remove from incubation.

A successful sterilization process ensures that the culture medium maintains its purple hue. Conversely, a defective sterilization process leads to a color change from purple to colorless.

Refer to the instructions provided with each presentation.

Precautions

Before using this product, please read the instructions thoroughly. Do not employ it for any purpose other than that specified; refer to the precautions and warnings section in the instructions.

Storage

The biological indicator must be stored at room temperature, protected from light and any sterilizing agents or other chemicals. Freezing is not permitted.

Useful Life

Shelf life of 24 months from the date of manufacture, provided that the specified storage conditions are maintained.

Responsible disposal

Dispose of pork biological indicators (BIs) in accordance with your country's health regulations. Positive BIs must be safely discarded and sterilized by autoclaving at 121°C for a minimum of 15 minutes, or at 132°C for 10 minutes in a gravity displacement steam sterilizer, or at 132°C for 4 minutes in a vacuum steam sterilizer.

Regulation

All activities conducted at EFELAB adhere to the ISO 13485 quality standard and comply with Good Manufacturing Practices (GMP) to guarantee product quality. This product meets the reference standards ISO 11138 (Parts 1 and 3) and IRAM 37102 (Parts 1 and 3).

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