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Self-Contained Biological Indicators for Ethylene Oxide (EO)

Product Overview

The TecSpores® Biological Indicator assesses sterilization efficacy through the use of spores *Bacillus atrophaeus* and a medium that transitions from fuchsia to translucent yellow upon the detection of bacterial growth.

Features

The *Bacillus atrophaeus* ATCC9372 strain is biocompatible for monitoring sterilization via Ethylene Oxide (EO). Its resistance to this process has established it as a representative organism for sterilization control.

Presentations

Self-Contained Biological Indicator Box: 100 units (BIE-C6) and 20 units (BIE-C620)

Exclusive access for healthcare professionals and institutions.

Rev05 14.04.26

   EFELAB S.R.L.

 (+5411) 4623-3030 | 4624-2400 | 4747-0000

 (+54) 11-5977-6578 |  www.efelab.com |  info@efelab.com

Purpose of Use

Position the biological indicator(s) (BI) within the sterilizer, conduct sterilization in accordance with established best practices, and subsequently remove the BI. Confirm the color change of the chemical indicator, then activate the BI alongside an unprocessed BI (serving as a positive control).

Incubate for 48 hours at 34-37°C, maintaining an upright position throughout the incubation period. Evaluate the results every 12 hours; upon obtaining a positive result, remove from incubation.

A successful sterilization process yields a permanent fuchsia hue in the culture medium. Conversely, a defective sterilization process leads to a color transition from fuchsia to translucent yellowish.

Refer to the instructions provided with each presentation.

Precautions

Before using this product, please review the instructions thoroughly.

Storage

The biological indicator must be stored at room temperature, protected from light and any sterilizing agents or other chemicals. Freezing is not permitted.

Useful Life

Shelf life of 24 months from the date of manufacture, provided that the specified storage conditions are maintained.

Responsible disposal

Dispose of biological indicators (BIs) in compliance with your country's health regulations. Positive BIs should be discarded using a safe method, such as autoclaving at 121°C for a minimum of 15 minutes, sterilizing at 132°C for 10 minutes in a gravity displacement steam sterilizer, or at 132°C for 4 minutes in a vacuum steam sterilizer.

Regulation

All activities conducted at EFELAB adhere to the ISO 13485 quality standard and comply with Good Manufacturing Practices (GMP) to guarantee product quality.

This product meets the applicable ISO standards.

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