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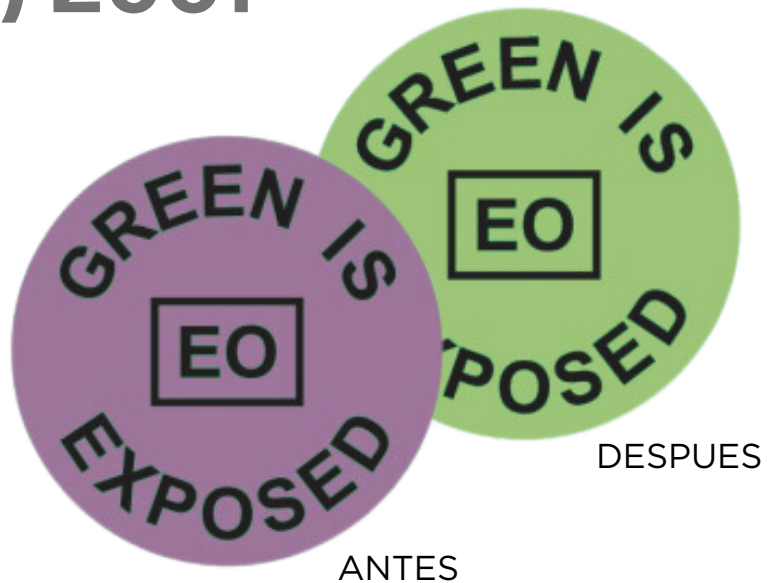
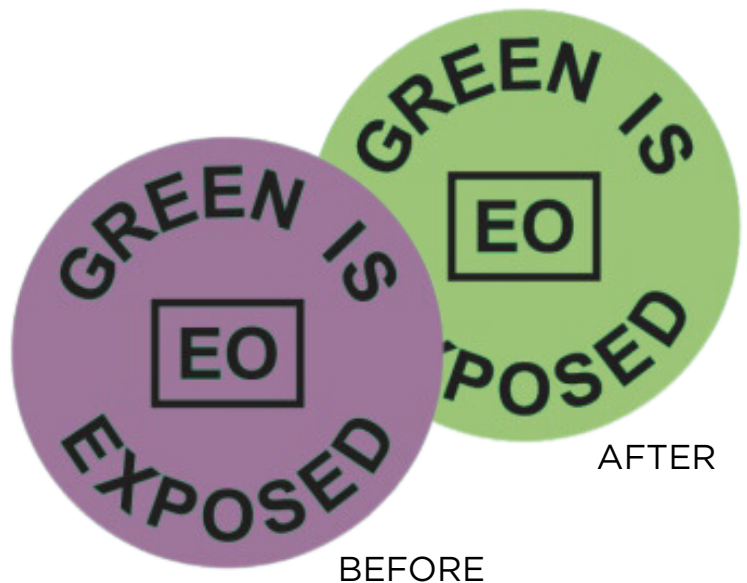
Sterilization indicator on circular self-adhesive label (Dot Label) 2001

Product Overview

Engineered to detect packages and materials subjected to sterilization, these self-adhesive labels exhibit a distinct color change following exposure to the requisite process conditions. Their circular design facilitates both manual and label-printing application, enhancing material control and traceability.

Features

- Circular self-adhesive label, 15 mm in diameter.
- Engineered for compatibility with standard labeling machines.
- Composed of 60-70 g paper featuring reinforced adhesive with strong initial tack, excellent adhesion, and cohesion.
- Full-surface printing utilizing sensitive ink, featuring the black inscription “GREEN IS EO EXPOSED,” which alters color following sterilization with ethylene oxide.

Presentation

A roll of 1000 labels housed in a laminated aluminum ZIPPER container (PET/AL/PE), sealed on three sides, with a thickness of 100 microns, featuring a doypack design and a heat-sealable top closure.

Sensitive dye

It responds to essential ethylene oxide sterilization parameters as outlined in ISO 11140-1, altering color upon exposure to 600 mg/L of EO (± 30 mg/L), at 37°C ($\pm 1^\circ\text{C}$), with 60% relative humidity ($\pm 10\%$) for a minimum duration of 30 minutes (± 30 sec).

How to Utilize

Ensure that the packages are clean and appropriately packaged. Insert a chemical indicator for integration and arrange them to facilitate gas circulation. Following sterilization with ethylene oxide, the indicator will transition from violet to green, thereby confirming that the requisite conditions have been achieved. If no color change is observed, the process has proven ineffective.

Precautions

This product should only be utilized for ethylene oxide sterilization. After exposure to ethylene oxide, the EFELAB self-adhesive label must be stored in a dark environment, as light can adversely affect the product.

Storage

Store in a cool location. Maintain dryness and protect from light.

Useful Life

The product has a shelf life of 60 months from the date of manufacture.

Regulation

All activities conducted at EFELAB adhere to the ISO 13485 quality standard and comply with Good Manufacturing Practices (GMP) to guarantee product quality.

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