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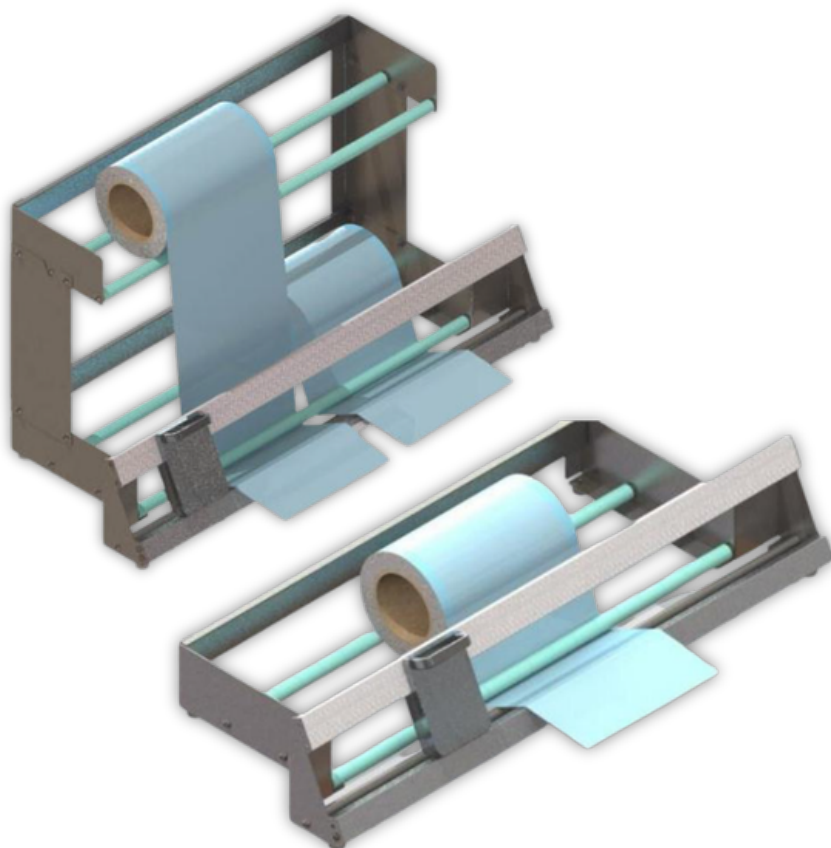
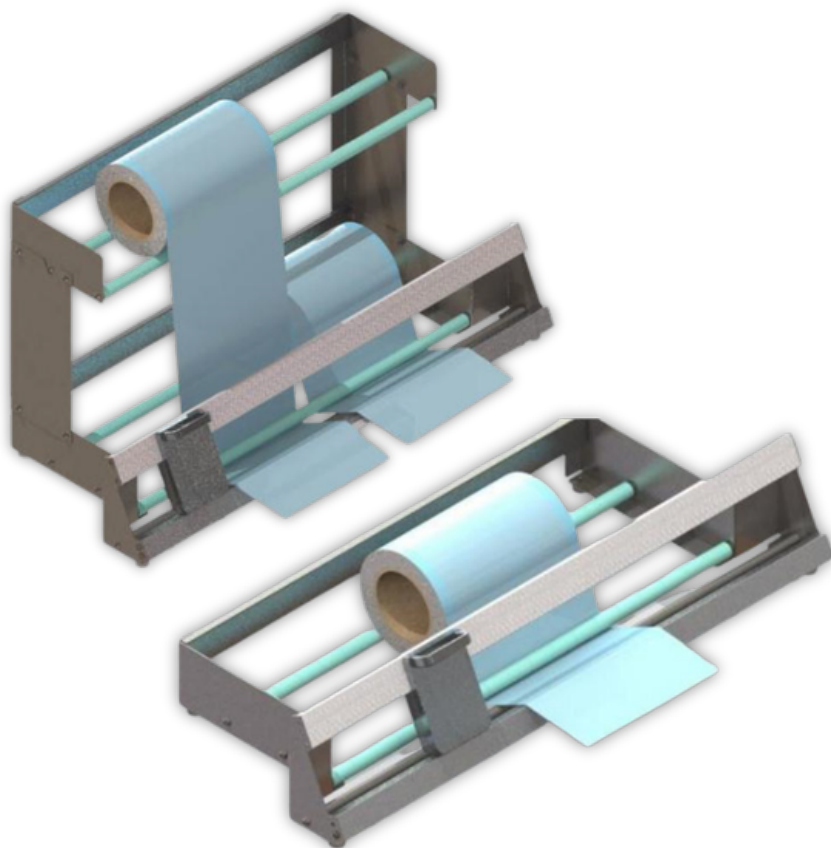
MEDICAL COIL CUTTER AND DISPENSER (CODE EF211 and EF212)

Product Overview


The Easyseal Codigo EF211 and EF212 manual dispensers are engineered for the rapid, accurate, and secure dispensing of medical packaging rolls and sleeves. Their innovative one-handed cutting mechanism facilitates continuous back-and-forth motion for cutting without the need to release the material, thereby enhancing work efficiency. Fabricated from aluminum alloy profiles and equipped with high-strength steel blades, they ensure straight, clean, and tear-free cuts.



Features

- **Continuous One-Hand Operation:** Enables the user to hold and pull the packaging while simultaneously operating the cutting carriage with one hand, eliminating the need to release the material or adjust grip between cuts, thereby optimizing the operator's efficiency.
- **Robust and Lightweight Chassis:** Constructed from high-stability aluminum alloy profiles.
- **High Precision Blade:** A high-hardness steel disc designed for a straight, burr-free cut, facilitating the easy replacement of consumables.
- **An ergonomic design** that enables the swift and unobstructed insertion and passage of the paper-plastic or Tyvek® strip through the cutting guide.
- **Multi-material compatibility:** Appropriate for flat and gusseted rolls of Paper-Plastic and Tyvek®.

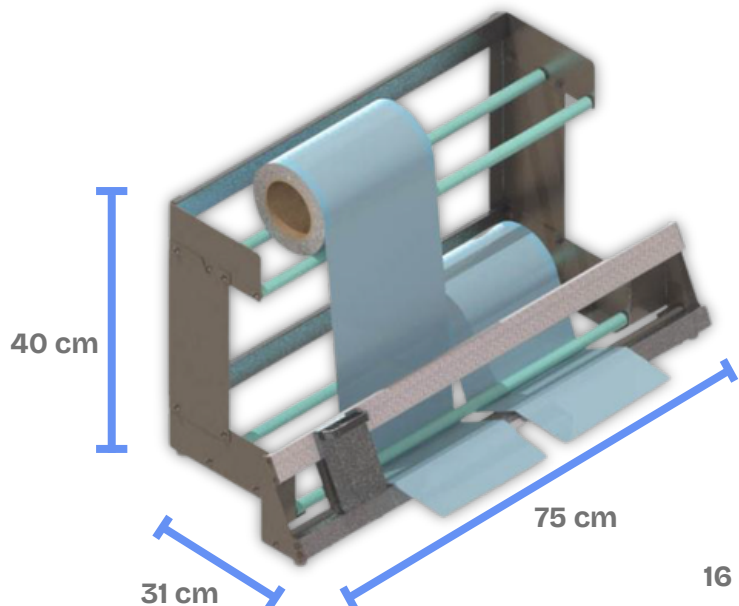Model: 212

Model: 211

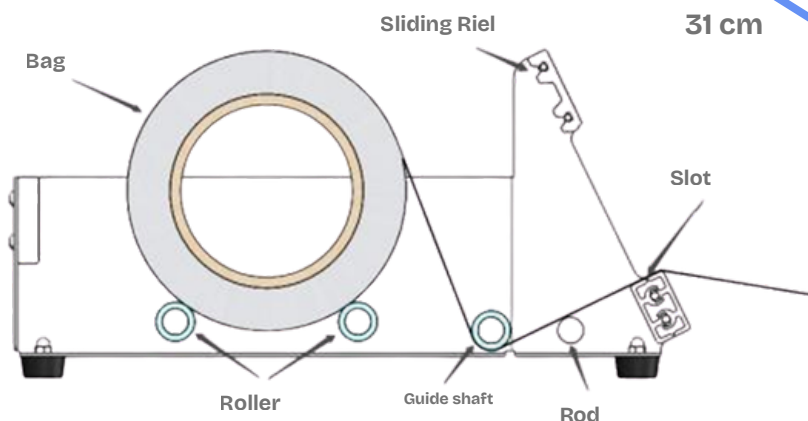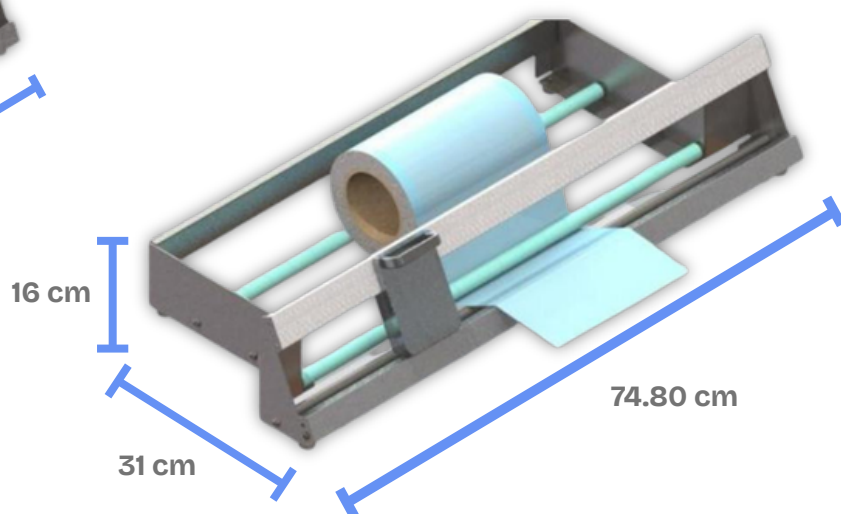How to Utilize

Installation: Position the equipment on a level surface in proximity to the sealer.

Loading: Position the coil between the rollers and thread the end through the guide shaft.

Cutting: Extend the spool to the desired length and maneuver the blade carriage. Pull once more without releasing the material and cut in the opposite direction for the subsequent unit.

Storage

Store in a clean, dry environment, shielded from moisture and corrosive substances. Retain in its original packaging during transportation or extended periods of inactivity.

Useful Life

EFELAB SRL provides a four-month warranty from the date of the invoice, which includes complimentary repairs and replacement parts. (Please note that blades are considered consumable items and are not included in the complimentary warranty.) Maintenance fees will be applicable, or a fee-based technical service will be provided in the following circumstances:

Equipment beyond the warranty period.

Damage or deformation resulting from external factors or improper use as outlined in the manual.

Utilizing inappropriate materials or employing non-genuine components not supplied by the manufacturer.

Disassembly, modification, or repair by unapproved personnel.

In the event of any issues, please contact EFELAB SRL or an authorized representative without delay.

Precautions

Exclusive Use: Intended solely for paper-plastic or Tyvek® rolls. Cutting other materials, such as cardboard, will result in blade damage.

Handling: Glide with consistent pressure along the guide, avoiding lateral force.

Safety: Avoid direct contact with the blade and ensure the cart is parked at one end at the conclusion of the day.

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